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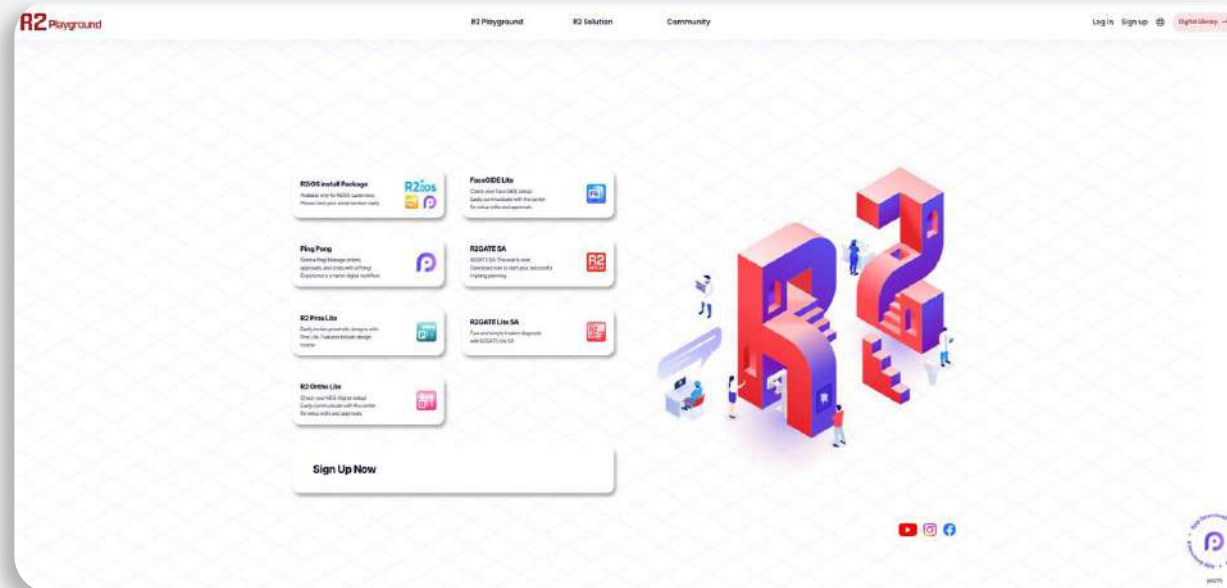
R2Playground.com

R2GATE™
by MEGA'GEN

Neuen Account erstellen

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R2Playground.com

R2GATE™
by MEGA'GEN

The screenshot shows the 'Sign up' page on the R2 Playground website. The page has a white background with a subtle pattern. At the top, there is a navigation bar with 'R2 Playground', 'R2 Solution', and 'Community' links. On the right, there are links for 'Log in', 'Sign up', and 'Sign up already'. The main heading is 'Sign up' with a small illustration of two people. Below this, a central box titled 'STEP 2 Email Registration Verification' contains the following fields and buttons:

- Email:** A label above two input fields.
- First Name:** A text input field with the placeholder 'Enter the email to be used as first name'.
- Last Name:** A text input field with the placeholder 'Enter the email to be used as last name'.
- Verify Email:** A button with the text 'Verify Email'.

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R2Playground.com

The screenshot shows a web interface titled "Your Files are ready! Please select a service." It features a form for patient information (Pt. Name, Patient name, Zip) and a grid of service options: R2 Guide, MEG Aligner, C&B, Face Guide, and QVD. A red arrow points from the "Nur Guide" label to the R2 Guide option. Another red arrow points from the "Behandlungsplan auswählen" label to the C&B option. A third red arrow points from the "Guide & Prothetik" label to the C&B option. At the bottom, there are two buttons: "Proceed to Order" and "Cancel Order". A red arrow points from the "Weiter zu den Bestelldetails" label to the "Proceed to Order" button.

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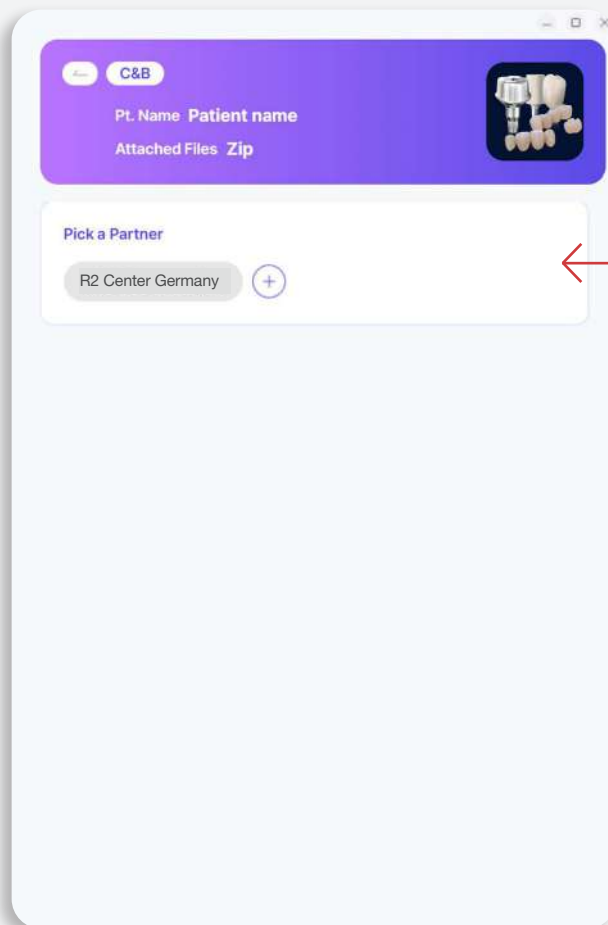
R2Playground.com

R2GATE™
by MEGA'GEN

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The screenshot shows a web application interface for placing orders. At the top, there is a purple header bar with a back arrow, a 'C&B' tab, and a thumbnail image of dental models. Below the header, there are two input fields: 'Pt. Name: Patient name' and 'Attached Files: Zip'. The main section is titled 'Pick a Partner' and contains a list of partners. The first partner, 'R2 Center Germany', is highlighted with a grey background and a plus icon to its right. A red arrow points from the text 'Wählen Sie das R2 Center' to this highlighted partner.

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R2Playground.com

R2GATE™
by MEGA'GEN

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The screenshot shows a web interface for placing an order. At the top, there's a purple header with 'C&B' and a patient name field. Below that is a 'Pick a Partner' section with a button for 'R2 Center Germany'. The 'Order details' section contains a grid of buttons for different dental services: Crown, Bridge, In/On-lay, Veneer, Custom abutment, PMMA Abutment, PFZ, ZrGEN, TiGEN, Cuff-TiGEN, and Provisional. There's a 'Show more' link below the grid. A checkbox for 'Design Only (Design service only.)' is present. The 'Comments' section has a text area for additional options and comments.

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R2Playground.com

R2GATE™
by MEGA'GEN

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The screenshot shows a web interface for placing an order. At the top, there's a purple header with a back arrow, 'C&B', 'Pt. Name Patient name', and 'Attached Files Zip'. Below this is a section for adding files, with a plus icon and text: 'Add file * You can upload up to 10 files. If you have any files related to the order, please attach them.' Below that is a 'Select Due Date' section with a date picker showing '06/07/2026' and a calendar for June 2026. The calendar has days of the week (SUN to SAT) and dates (1 to 30). The date 29 is highlighted in blue.

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You can upload up to 10 files

+

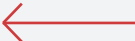
If you have any files related to the order, please attach them.

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Please see below for details

C&B

Pt. Name	Patient name
Attached Files	Zip(1) 
Order Option	-
Partner	MGG EU education
Due Date	2026.07.28
Comments	
Details about case	

Orderer information (ID, clinic name, representative, contact, address, etc.) will be shared with the partner for order processing.

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MegaGen F.D. GmbH