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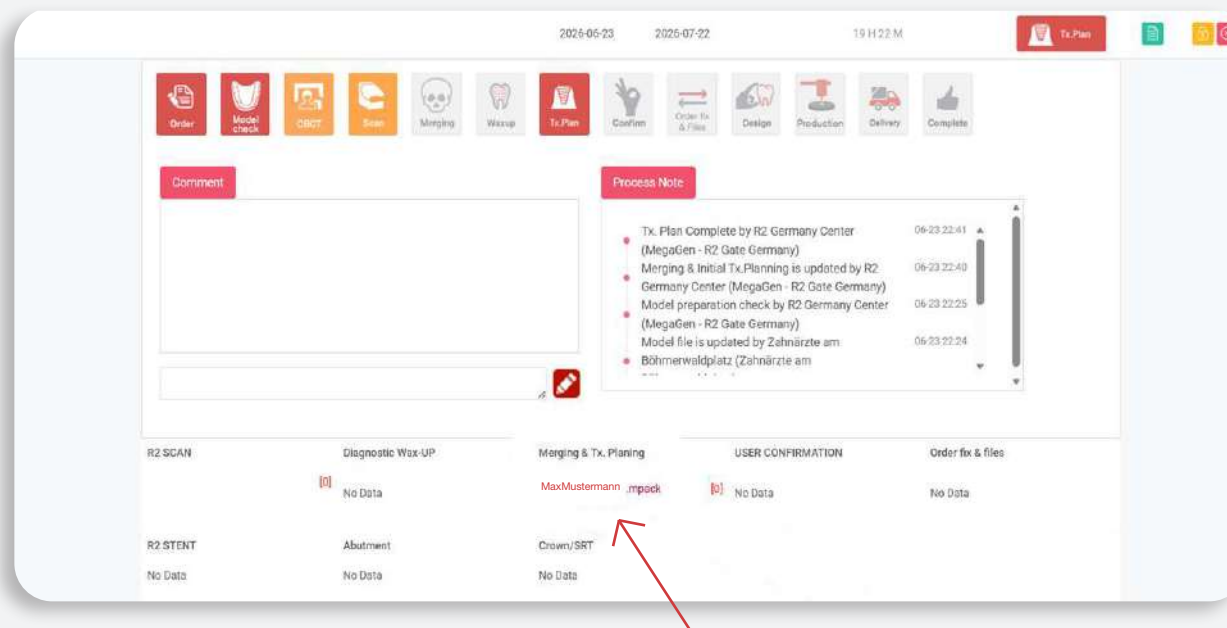
The screenshot displays the R2Gate.com web application interface. A central 'USER CONFIRMATION' modal is open, prompting the user to 'Upload the final plan (project file)'. It features a 'COMMENT' text area where the text 'Op Termin ist am 05.07.2024' has been entered, highlighted by a red arrow. The background interface includes a top navigation bar with icons for 'Order', 'Model', 'Scan', 'Scan', and 'Merging'. Below this, there's a 'Comment' section with a text input field. The main content area shows a table with columns for 'R2 SCAN', 'Diagnostic Wax-UP', 'R2 STENT', 'Abutment', and 'Crown/SRT'. The table contains 'No Data' entries. The right sidebar has buttons for 'Production', 'Delivery', and 'Complete'. The bottom of the screen shows a 'Confirm' button and a 'Close' button.

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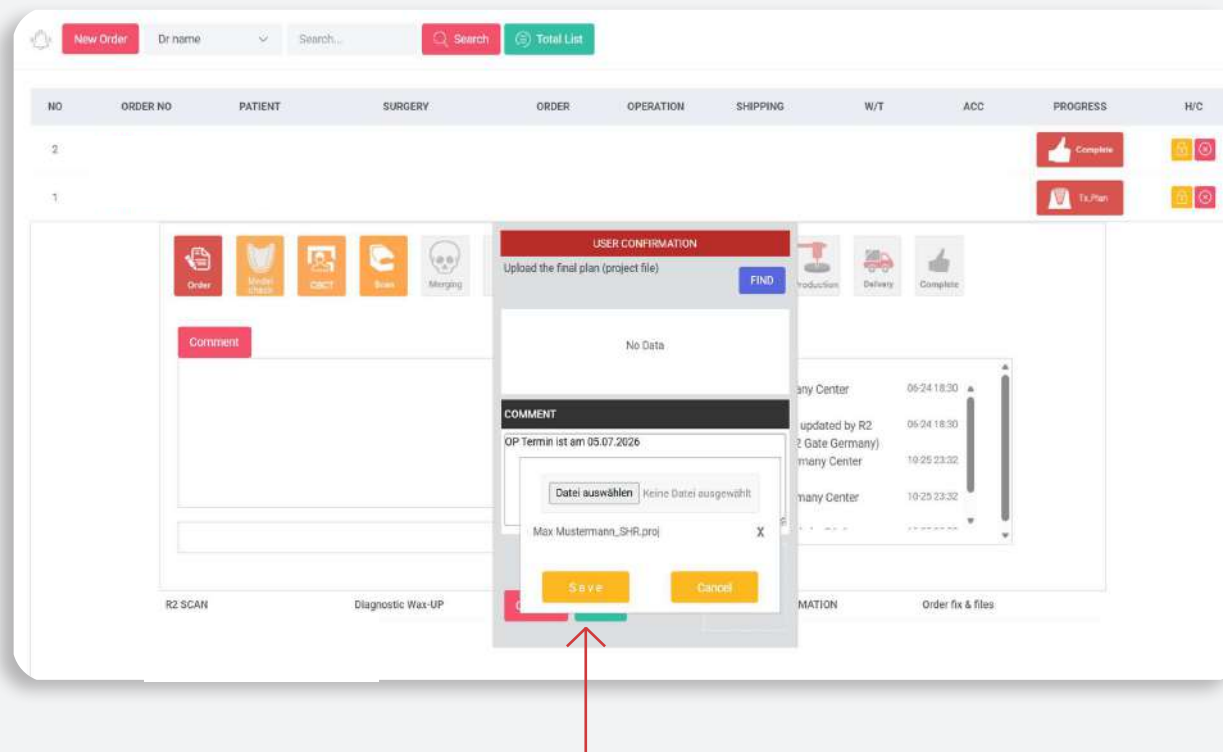
The screenshot displays the R2Gate.com web interface. At the top, there is a navigation bar with a 'New Order' button, a dropdown for 'Dr name', a search bar, and a 'Total List' button. Below this is a table with columns: NO, ORDER NO, PATIENT, SURGERY, ORDER, OPERATION, SHIPPING, W/T, ACC, PROGRESS, and H/C. The table contains two rows of data. A modal dialog box titled 'USER CONFIRMATION' is open in the center. It has a red header and contains the following elements: a text input field for 'Upload the final plan (project file)' with a 'FIND' button; a text input field containing 'Max Mustermann_SHR.proj'; a 'COMMENT' section with a text area; and 'Confirm' and 'Close' buttons at the bottom. A red arrow points to the 'Confirm' button. The background shows a sidebar with icons for 'Order', 'Upload plan', 'CBCT', 'Scan', and 'Merging', and a main area with a 'Comment' section and a 'Diagnosis & Tx. planning' section.

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MegaGen F.D. GmbH